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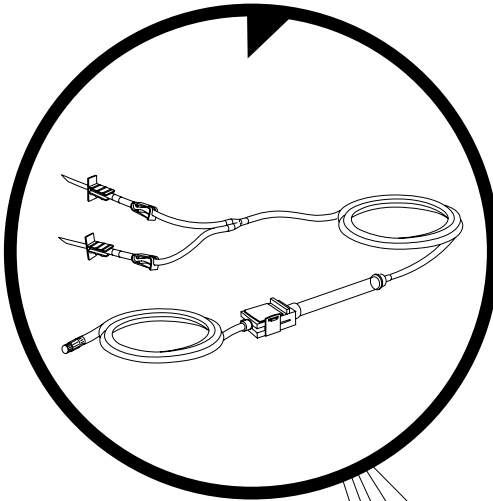
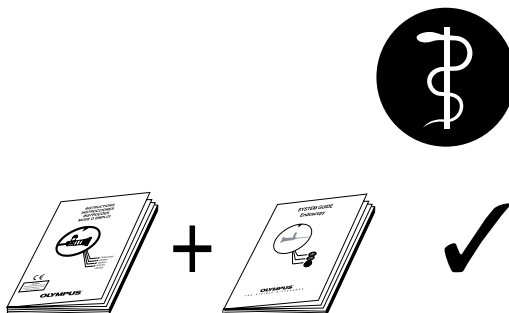
General Information

Intended use.

Tubing set for performing irrigation or distension in endoscopic applications. Do not use it for any other purpose.

Medical use. This product is to be used only in a medical facility by trained medical personnel.

NOTE!

Instruction manual set.

The complete instruction manual set for this product consists of this product-specific instruction manual (delivered together with this product) and the system-related instruction manual, entitled "Olympus Endoscopy System Guide" (supplied separately). Immediately contact an Olympus representative if either manual is missing or lost.

Keep the instruction manual set in a safe, accessible location.

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**WARNING!**

Read all instruction manuals thoroughly. Before use, read this manual, the Olympus Endoscopy System Guide, and the manuals for all other equipment which will be used during the procedure.

An insufficient understanding of the dangers, warnings, cautions, and information in these manuals can result in death, serious injury, or equipment damage.

CE 0197

Medical Device Directive (Europe).

This product complies with the requirements of Directive 93/42/EEC concerning medical devices.

Manufacturer:

W.O.M. WORLD OF MEDICINE GmbH
Salzufer 8
10587 Berlin
Germany

Distributor:

Olympus Winter & Ibe GmbH
Kuehnstr. 61
22045 Hamburg
Germany

**Upon Delivery****Inspect immediately.**

- Make sure that the shipping case contains all of the items listed below.
- Contact your Olympus representative or an authorized service center if any items are missing or damaged.

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Before Use

WARNING!

Infection control risk (A4055 only). Properly reprocess the product before first and each subsequent use following the instructions in this manual and in the Olympus Endoscopy System Guide. Improper and/or incomplete reprocessing can cause infection of the patient and/or medical personnel.

WARNING!

Sterilized product (A4054, A4056 only). This product is delivered in a sterilized condition. Use only if the package is undamaged. Only open the package immediately before use.

Inspection

Inspecting the product. Visually inspect the product. Make sure that it has:

- no cracks

Damaged product. If the product is damaged or does not function properly, contact an Olympus representative or an authorized service center.

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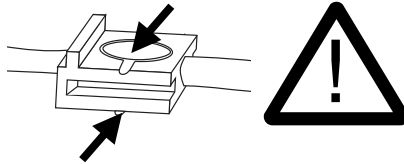
Tubing Connection

Select a compatible tubing set.

- Refer to the pump's instruction manual.
- Tubing set A4054, A4056: Sterile, intended for single-use only. Do not use if the packaging is damaged. Do not attempt to reuse or reprocess.
- Tubing set A4055: Non-sterile, intended to be used several times. Sterilize before use.


CAUTION!

Risk of damage. Make sure that the tubing set is connected without being kinked or pinched.

CAUTION!

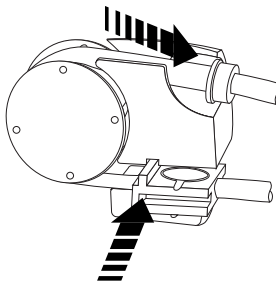Risk of damage. Do not insert the tubing set's pump section while filled. Otherwise the pressure chamber's membranes may be damaged.

Insert the tubing set's pump section.

- Before first use, remove the light-blue transport safeguard from the pressure chamber.
- Insert the pressure chamber into the lower receiver of the pump head. The pressure chamber must click into position if correctly inserted.
- Stretch the tubing set's flexible pump section around the roller wheel and hook onto the upper receiver of the pump head.

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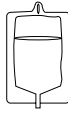
§

WARNING!

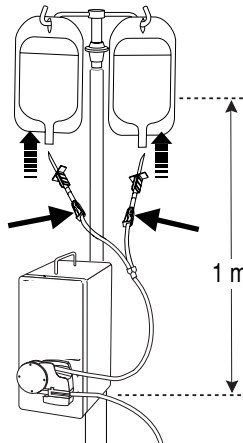
Irrigation fluid. The irrigation fluid must comply with national pharmaceutical regulations.

WARNING!

Distension medium. The distension medium must comply with national pharmaceutical regulations.

CAUTION!

Risk of implosion. Use only flexible irrigation bags.
Do not use glass bottles.

Connect irrigation bags.

- Position two irrigation bags containing an appropriate irrigation fluid on a suitable stand at a distance of one meter above pump level. Refer to the pump's instruction manual.
- Close the tubing clamps.
- Insert the cannulas into the irrigation bags.

Connect a suction/irrigation tube or an endoscope.

- Refer to the pump's instruction manual.
- Open one or both tubing clamps.

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Use

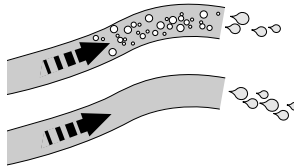
Before use (A4055 only). Make sure that the product has been properly re-processed, inspected, and tested.

Before use (A4054, A4056 only). Make sure that the product has been properly inspected and tested.

Preparation

Displace air from the tubing system.

- Open the suction/irrigation tube's or endoscope's valve: Displace all air bubbles by letting irrigation fluid/dis-tension medium flow through the tubing system prior to each use.

Procedure

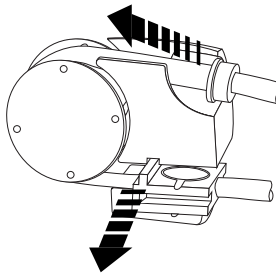
Procedure.

- Refer to the pump's instruction manual.

After Use

Finish the procedure.

- Refer to the pump's instruction manual.
- Immediately after use, remove the tubing set from the pump and from the suction/irrigation tube or endoscope.

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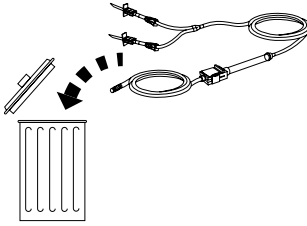
Disposal (A4054, A4056 only).

- Dispose of the product after use.
Do not reuse or attempt to resterilize!

Reprocessing (A4055 only).

- Immediately bring the product to the reprocessing area.
- Reprocess the product following the instructions in this manual and in the Olympus Endoscopy System Guide.

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WARNING!

Infection control risk.

Properly reprocess the product before first and each subsequent use following the instructions in this manual and in the Olympus Endoscopy System Guide. Improper and/or incomplete reprocessing can cause infection of the patient and/or medical personnel. Observe the following instrument-specific instructions.

Information in this section. The following reprocessing information supplements that in the Olympus Endoscopy System Guide.

However, if the information given in this section differs from that in the Olympus Endoscopy System Guide, use the information given here.

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	A4055
① Manual or automatic cleaning	+
② Ultrasonic cleaning	+
③ Disinfection or high-level disinfection	+
④ Steam sterilization (autoclaving)	+
⑤ Heat sterilization (dry)	-
⑥ Gas sterilization (ETO)	+
⑦ Gas sterilization (formaldehyde)	+
⑧ STERRAD 50/100/100S	-
⑨ STERIS	-

Compatible reprocessing methods.

This product is compatible with the reprocessing methods listed in the table. For details of the procedures, refer to the Olympus Endoscopy System Guide.

- + compatible
- incompatible

- ① Manual or automatic cleaning
- ② Ultrasonic cleaning
- ③ Disinfection or high-level disinfection
- ④ Steam sterilization (autoclaving)
- ⑤ Heat sterilization (dry)
- ⑥ Gas sterilization (ETO)
- ⑦ Gas sterilization (formaldehyde)
- ⑧ STERRAD 50/100/100S
- ⑨ STERIS

CAUTION!

Risk of damaging the product.

Do not use incompatible reprocessing methods. Product damage may occur.

Disinfection

CAUTION!

Risk of damaging the product.

Do not leave the silicone tubing in the disinfection solution for more than 30 min. Silicone absorbs various disinfection solutions and is damaged when subsequently sterilized in a steam autoclave.

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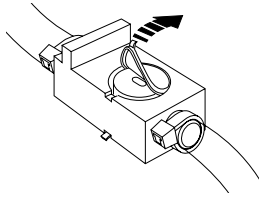
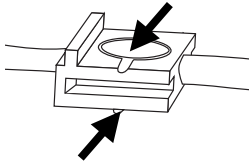
Sterilization

CAUTION!

Sterilization of the tubing. The number of sterilization cycles for the tubing set is limited (see packaging). Do not exceed the number of sterilization cycles indicated by the manufacturer.

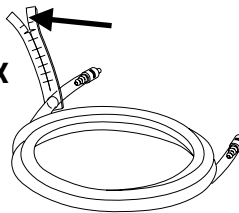
Number of sterilization cycles.

- Record the number of sterilization cycles by means of the tear-off straps attached to the tubing set. If all tear-off straps have been removed, the tubing set must neither be sterilized again nor reused.

Storage. Store this product in a clean, dry area at room temperature. Do not expose directly to sunlight. Do not expose to high temperatures.

20 x

OLYMPUS

7035794_8-0
2014-09-23
en, de, cs, el

© Copyright 2014 Olympus Winter & Ibe GmbH