INSTRUCTIONS  
GEBRAUCHSANWEISUNG  
INSTRUCCIONES  
INSTRUÇÕES  
MODE D'EMPLOI  
ISTRUZIONI PER L'USO  
GEBRUIKSAANWIJZING

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CE 0473

**Caution:**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**OLYMPUS®**

**GB**

## General Information

**Intended use.** This product has been designed for use with insufflators during endoscopic interventions. The use of this filter minimizes the risk of fluid flowing into the unit and prevents contamination of the unit.  
Do not use it for any other purpose.

**Medical use.** This product is to be used only in a medical facility by trained medical personnel.

**Safety advisory.** Follow all warnings and cautions contained in this manual and in the Olympus Endoscopy System Guide.

### CAUTION!

**Sterilized single-use product.** This product is delivered in a sterilized condition. Do not use the product if the packaging is damaged. Do not open the packaging until immediately before use.

This product is intended for single use only. Dispose of the product after use. Do not reuse!

**Storage.** Store this product in a clean, dry area at room temperature. Do not expose directly to sunlight. Do not expose to high temperatures.

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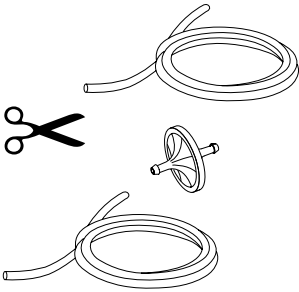
**USA**

**Insert the filter.**

- Cut through the insufflation tubing near the insufflator under sterile conditions.
- Insert the filter.  
One of the following markings indicates the insufflator's side of the A5623 filter:  
- Green dot

**CAUTION!**

**Reduced insufflation capacity.**

The filter reduces the insufflator's flow rate.

Refer to the insufflator's instruction manual.

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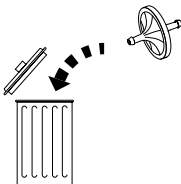
**After use**

**Single use.**

- The product is intended for single use only.  
Dispose of the product after use.  
Do not reuse!

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© Copyright 2014 Olympus Winter & Ibe GmbH
GVS Filter Technology UK Ltd