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Siemens Healthcare, s.r.o.

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Specification sheet

Sysmex CS-5100 System

Break through the routine with PSI technology

The Sysmex® CS-5100 System is a fully automated, computer-interfaced coagulation analyzer for in vitro diagnostic use that helps you to transform patient care through highly reliable, rapid sample processing. Optional connectivity to Aptio® Automation in various customized configurations including single-discipline and multi-disciplinary configurations supports increased workforce productivity in high-volume labs. The system helps to minimize hands-on time through optimized reagent vial management, a large disposable inventory, and continuous connectivity to a water supply. New multiwavelength analysis in combination with PSI™ technology for checking pre-analytical variables enable reliable results on the first run.

siemens-healthineers.com/hemostasis

Reagent test menu

PT	Dade® Innovin®, Thromborel® S
APTT	Dade Actin®, Dade Actin FS, Dade Actin FSL, Pathromtin® SL
Fibrinogen	Dade Thrombin
Thrombin time	Test Thrombin, Thromboclotin®
Batroxobin time	Batroxobin
Factor deficiency	Factor II, V, VII, VIII, IX, X, XI, XII
Lupus anticoagulant	LA1 screening, LA2 confirmation
Protein C pathway	Protein C, ProC® AcR,* ProC Global,* F V Leiden, Protein S Ac, INNOVANCE® Free PS Ag
Anticoagulant therapy management	INNOVANCE Anti-Xa,* INNOVANCE Heparin, INNOVANCE DTI,* direct oral anticoagulant testing upon request
Antithrombin III	Berichrom® Antithrombin III (A), INNOVANCE Antithrombin
D-dimer	INNOVANCE D-Dimer
Von Willebrand factor	vWF Ag,* INNOVANCE VWF Ac,* BC von Willebrand
Chromogenic	Factor VIII Chromogenic, BIOPHEN Factor IX,*† Berichrom Factor XIII Chromogenic,* Berichrom C1-Inhibitor,* Berichrom α2-Antiplasmin, Berichrom Plasminogen, Berichrom Protein C
Platelet aggregation	ADP,*† Epinephrine,*† Ristocetin,*† Collagen,*† Arachidonic acid*†

Reagent names are abbreviated. Please refer to instrument-specific reagent application/reference guide for more information. Product availability varies by country.

*The products/features/applications mentioned here are not commercially available in all countries and are subject to local regulations. Their future availability cannot be guaranteed. Not available for sale in the U.S.

†HYPHEN BioMed application.

Sysmex CS-5100 System specifications

Measurement	
Principle	Photo-optical, continuous, sequential based on change in the transmitted light emitted from the sample with added reagent
Method/channels	20 single channels for clotting, chromogenic, immunoassay, and aggregation methods
Measurement channels	20 wells (mixing function using stir bars possible for 8 wells)
Source lamp	Halogen lamp for measurements at 340, 405, 575, 660, and 800 nm wavelengths
Analysis mode	Normal and micro-sample modes
Analysis method	Dilution analysis; multidilution analysis (MDA); automatic reanalysis (redilution analysis, reanalysis, reflex testing); mixing testing; platelet aggregation testing;† clot waveform analysis (CWA)‡
Time resolution	Sampling can be performed for up to 1800 seconds at intervals of 0.1 second
Measuring time	Up to 1800 seconds for each parameter
Incubation	36 wells
Number of user-definable (open) channels	80,000
Sample handling	
Type of sample	Primary tubes and/or sample cups
Sampling mechanism	Automated sample and standard predilution
Pre-analytical variables check (PSI)	Assay-based qualitative check for hemolysis, lipemia, icterus, and proper fill of primary tube
Traceability of results	Operator name, test date and time, reagent lot information, reagent elapsed time onboard, HIL and sample-volume checks, test reaction position, testing and reagent table temperatures, test protocol number, dilution ratio, QC performed date, calibration curve identification, and maintenance and error logs
Carrier system	Continuous-access sample racks with 10-tube capacity
Maximum load	100 samples; 10 racks with 10 samples per rack
STAT sample loading	Five priority positions
Storage temperature	Room temperature
Racks	10 sample racks, continuous loading; various specific tube holders and adapters available
Handling	Flexible mix of various capped and/or uncapped sample tubes and 4 mL conical sample cup

Sample handling (continued)			
Primary sample probe	Liquid-level sensing, crash protection, clog detection, liquid surface verification		
Automatic sample predilution	Samples can be diluted (0 to 19:1 to 259) for multidilution analysis (MDA) and mixing tests and can be retained for auto repeat until results are available		
Cap piercing	Pierces caps on primary tubes		
Reagent handling			
Reagent recognition	Internal reagent bar-code identification		
Dispensing accuracy	Volume dispensed	Reproducibility	Accuracy
Reagent probe	50 µL	CV 3% or less	48.0 ±3.0 µL
	100 µL	CV 1% or less	99.0 ±2.0 µL
Sample probe	10 µL	CV 5% or less	9.5 ±1.0 µL
	100 µL	CV 1% or less	98.5 ±2.0 µL
Volume	Reagent dispensing pipette: 20–200 µL Sample dispensing pipette: 20–250 µL (combined diluent and sample) Aspirated volumes of sample and diluent are in the range of 4–150 µL		
Dispensing mechanism	5 probes: 3 heated for reagents and 2 for samples, controls, and calibrators. 3 catchers		
Loading system	Manual placements of reagents, removable reagent trays		
Reagents onboard	40 reagent/control positions and 5 buffer/rinse solution positions		
Storage temperature	40 cooled reagent positions onboard at 10°C ±2°C		
Handling	Flexible mix of reagent positions within the reagent table; various adapters available		
Mixing position	Up to 10 positions available on the reagent table		
Storage capacity onboard	1890 tests average; 3000 tests maximum		
Onboard stability	System keeps reagents cooled 24 hours/day, 7 days/week; anti-evaporation caps support long onboard stability		
Reagent inventory management	Tracks number of tests remaining, lot onboard stability, vial type, set date and time, and expiration date		
Throughput [§] (tests/hour, approx.)			
PT	400 (single-parameter analysis)		
APTT	400 (single-parameter analysis)		
PT/APTT	400 (simultaneous analysis)		
PT/APTT/AT/DD	282 (simultaneous analysis) Results for normal mode processing		

†HYPHEN BioMed application.

‡Research use only.

§Throughput values were determined by the time to first result; processing capability varies depending on the reagent used. Throughput values stated above were determined using Siemens Healthineers study protocol with PT (Thromborel S Reagent), APTT (Pathromtin SL Reagent), INNOVANCE D-Dimer Reagent, and AT (INNOVANCE AT Reagent) test applications.

Operation	
Access mode	Continuous random access (samples, reaction tubes)
Calibration	2–12 point calibration curves with maximum of 5 repeated analyses per point and up to 10 calibration curves; one reagent lot group
Calibration curve	250 user-defined parameters
View calibration	Graphical display of calibration curves from up to 10 different reagent lots/parameters
Auto calibration/ auto QC	User-defined time interval or with new reagent vial
Temperature control	37°C ±0.5°C for measuring channels, incubation area, reagent probe
Sample and abnormal reaction monitoring	Coagulation curve abnormalities; overreaction to antigen; interfering assay-based HIL; sample volume check; analysis value error monitoring (checks for deviations based on mark limit values, report limits, and the results of repeated analyses)
Automation connectivity	Point-in-space connectivity; single-discipline and multi-disciplinary automation is supported by Aptio Automation, FlexLab™ Automation, and other third party solutions.
Reaction tubes	
Type	Single reaction tubes
Loading	Automatic continuous access; 1000 reaction tubes onboard
Reaction tube stirring	Yes
System fluid	
Cleaning and rinsing	Washing solutions onboard
System containers	20 L containers for water (rinse) and waste (optional)
Computer/printer	
Workstation	PC
Display	19-inch wide-screen LCD display with touch operation
Printer	Graphic printer (optional)
Input devices	Touchscreen, keyboard, and mouse, 2D bar-code reader
Data storage	About 10,000 samples, with a maximum of 60 results per sample
Onboard maintenance logs	Scheduled and automatically monitored routine maintenance activities via software
Software	
LIS interface	Sysmex CA-1500, CS-2500, PC-DPS(C), ASTM1381-95/1394-95, ASTM1381-02/1394-97
Host connection	Bidirectional RS-232C serial port or via Ethernet-TCP/IP
Operating system	MICROSOFT WINDOWS 10-based
System documentation	Instruction manual and reference guide online

Power supply	
Operating voltage	Main unit: 200–240 V Pneumatic unit: 100–117 V/220–240 V
Power consumption	Main unit: ≤1700 VA Pneumatic unit (processing): ≤280 VA
Main frequency	50–60 Hz
Environmental conditions	
Operating temperature	15–30°C
Environmental humidity	30–85% (no condensation except on the reagent table)
Atmospheric pressure	70–106 kPa
Waste heat	Approx. 4000 BTU/h (1040 Kcal/h)
Noise level	60 dB or below (excluding sudden noise that stops within 5 seconds and alarms)
Dimensions	
Main unit	Approx. 1030 (W) x 1150 (D) x 1280 (H) mm 40.6 (W) x 45.3 (D) x 50.4 (H) in.
Pneumatic unit	Approx. 280 (W) x 355 (D) x 400 (H) mm 11.1 (W) x 14.1 (D) x 15.7 (H) in.
Information processing unit (IPU) cart	Approx. 546 (W) x 891 (D) x 1,195 (H) mm 21.5 (W) x 35.1 (D) x 47.1 (H) in.
Weight	
Main unit	Approx. 278 kg /612.9 lb
Pneumatic unit	Approx. 17 kg /37.5 lb
Information processing unit (IPU) cart	Approx. 67 kg /147.7 lb
Quality control	
X-control, Levey-Jennings control, multi-rule monitoring (Westgard rule). Maximum of 1200 plots x 750 files can be saved.	
Compliance	
Safety standards	
IEC 61010-1:2010, IEC 61010-2-081:2015, IEC 61010-2-101:2015, IEC 60825-1:2014, EN 61010-1:2010, EN 61010-2-081:2015, EN 61010-2-101:2017, EN 60825-1:2014, UL 61010-1:2012, UL 61010-2-081:2015, UL 61010-2-101:2015, CAN/CSA-C22.2 no. 61010-1-12, CAN/CSA-C22.2 no. 61010-2-081:15, CAN/CSA-C22.2 no. 61010-2-101:15, CAN/CSA-E60825-1:15	
EMC standards	IEC 61326-2-6:2012, EN 61326-2-6:2013
Ordering information	
Product description	Catalog number
Sysmex CS-5100 System	10709128
Sysmex CS-5100 System kit	10713586

For more information, please contact your Siemens Healthineers representative or visit siemens-healthineers.com/hemostasis.

At Siemens Healthineers, we pioneer breakthroughs in healthcare. For everyone. Everywhere. By constantly bringing breakthrough innovations to market, we enable healthcare professionals to deliver high-quality care, leading to the best possible outcome for patients.

Our portfolio, spanning from in-vitro and in-vivo diagnostics to image-guided therapy and innovative cancer care, is crucial for clinical decision-making and treatment pathways. With our strengths in patient twinning, precision therapy, as well as digital, data, and artificial intelligence (AI), we are well positioned to take on the biggest challenges in healthcare. We will continue to build on these strengths to help fight the world's most threatening diseases, improving the quality of outcomes, and enabling access to care.

We are a team of 66,000 highly dedicated employees across more than 70 countries passionately pushing the boundaries of what's possible in healthcare to help improve people's lives around the world.

**This brochure is intended for use outside the U.S. only (OUS).
Display or promotion to U.S. audiences is prohibited.**

Actin, Aptio, Berichrom, Dade, INNOVANCE, Innovin, Pathromtin, ProC, Thromboclotin, Thromborel, PSI, and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. Sysmex is a trademark of the Sysmex Corporation. Aptio Automation is manufactured by Inpeco SA and is exclusively distributed by Siemens Healthcare Diagnostics Inc. FlexLab is a trademark of Inpeco SA. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Sysmex CS-5100 System refers to Automated Blood Coagulation Analyzer CS-5100.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
[REDACTED]
siemens-healthineers.com

Legal Manufacturer

Sysmex Corporation
1-5-1 Wakinohama-Kaigandori,
Chuo-ku,
Kobe 651-0073, Japan

for the Products

PSI technology, Sysmex CA-1500 System, Sysmex CS-2500 System, Sysmex CS-5100 System

Legal Manufacturer

HYPHEN BioMed
155 rue d'Eragny
95000 Neuville-sur-Oise
France

for the Products

ADP, Epinephrine, Arachidonic Acid, Ristocetin, Collagen

Distributed by

Siemens Healthcare Diagnostics Products GmbH
Laboratory Diagnostics
Emil-von-Behring-Strasse 76
35041 Marburg, Germany

Legal Manufacturer

for all other products
Siemens Healthcare Diagnostics Products GmbH
Laboratory Diagnostics
Emil-von-Behring-Strasse 76
35041 Marburg, Germany

Legal Manufacturer

Inpeco SA
Via Torraccia 26
6883 Novazzano
Switzerland

for the Products

Aptio Automation, FlexLab Automation

Specification sheet

Sysmex CN-3000 and CN-6000 Systems

Where size, speed, intelligence, and flexibility combine to accelerate hemostasis workflows

The Sysmex® CN-3000 and CN-6000 Hemostasis Systems* offer industry-leading throughput in the smallest footprint on the market.† With intelligent technology and AI-driven predictive analytics, they streamline workflow through improved sample and error management. They can even calculate the day's testing reagent requirements to reduce workflow interruptions. And for labs seeking to automate testing, they offer multiple automation options ranging from automated hemostasis workcells‡ to integration with track-based automation.

[siemens-healthineers.com/
sysmex-cn-systems](https://www.siemens-healthineers.com/sysmex-cn-systems)

Reagent test menu	
PT	Dade® Innovin®, Thromborel® S
APTT	Dade Actin®, Dade Actin FS, Dade Actin FSL, Pathromtin® SL
Fibrinogen	Dade Thrombin
Thrombin time	Test thrombin, Thromboclotin®
Batroxobin time	Batroxobin
Factor deficiency	Factor II, V, VII, VIII, IX, X, XI, XII
Lupus anticoagulant	LA1 screening, LA2 confirmation
Protein C pathway	Protein C, ProC® AcR, * ProC Global, * F V Leiden, Protein S Ac, INNOVANCE® Free PS Ag
Anticoagulant therapy management	INNOVANCE Heparin (LMWH, UFH), INNOVANCE Anti-Xa (LMWH, UFH, rivaroxaban, apixaban), * INNOVANCE DTI (dabigatran), * other direct oral anticoagulant testing upon request*§
Antithrombin III	Berichrom® Antithrombin III (A), INNOVANCE Antithrombin
D-dimer	INNOVANCE D-Dimer
Von Willebrand factor	vWF Ag, * INNOVANCE VWF Ac, * BC von Willebrand
Chromogenic	Berichrom Factor VIII Chromogenic, Berichrom Factor XIII Chromogenic, * Berichrom C1-Inhibitor, * Berichrom α2-Antiplasmin, Berichrom Plasminogen, Berichrom Protein C, Factor IX Chromogenic*§
Platelet aggregation	ADP,*§ Epinephrine,*§ Ristocetin,*§ Collagen,*§ Arachidonic acid*§

Reagent names are abbreviated. Please refer to instrument-specific reagent application/reference guide for more information. Product availability varies by country. *Not available for sale in the U.S. The products/features mentioned here are not commercially available in all countries. Their future availability cannot be guaranteed.

†As of March 2021. Sysmex CN-6000 System throughput: 400 PT/APTT; Sysmex CN-3000 System throughput: 215 PT/APTT. Throughput evaluated against systems in the same volume class; footprint compared to competitor systems as listed on <https://www.captodayonline.com/>.

‡Under development. Not available for sale. Future availability cannot be guaranteed.

§ HYPHEN BioMed reagents.

Sysmex CN-3000 and CN-6000 System specifications

Measurement							
Principle	Photo-optical, continuous, sequential based on change in the transmitted light emitted from the sample with added reagent						
Method/channels	<table border="1"> <thead> <tr> <th>CN-3000</th> <th>CN-6000</th> </tr> </thead> <tbody> <tr> <td>13 single channels</td> <td>26 single channels</td> </tr> <tr> <td colspan="2">For clotting, chromogenic, immunoassay, and aggregation methods</td> </tr> </tbody> </table>	CN-3000	CN-6000	13 single channels	26 single channels	For clotting, chromogenic, immunoassay, and aggregation methods	
CN-3000	CN-6000						
13 single channels	26 single channels						
For clotting, chromogenic, immunoassay, and aggregation methods							
Measurement channels	<table border="1"> <thead> <tr> <th>CN-3000</th> <th>CN-6000</th> </tr> </thead> <tbody> <tr> <td>13 wells (mixing function using stir bars possible for 4 wells for aggregation methods)</td> <td>26 wells (mixing function using stir bars possible for 8 wells for aggregation methods)</td> </tr> </tbody> </table>	CN-3000	CN-6000	13 wells (mixing function using stir bars possible for 4 wells for aggregation methods)	26 wells (mixing function using stir bars possible for 8 wells for aggregation methods)		
CN-3000	CN-6000						
13 wells (mixing function using stir bars possible for 4 wells for aggregation methods)	26 wells (mixing function using stir bars possible for 8 wells for aggregation methods)						
Source lamp	LED lamp with 5-year lifespan for measurements at 340, 405, 575, 660, and 800 nm wavelengths. Measurement of up to five different wavelengths simultaneously. Flexible detection method switches between main and sub-wavelengths and makes light intensity adjustments for sample resulting, even with high hemolytic, icteric, or lipemic (HIL) interferences.						
Analysis mode	64 aliquot positions onboard for normal and micro-sample modes, regardless of automation connectivity						
Analysis method	Dilution analysis; multidilution analysis (MDA); automatic reanalysis (redilution analysis, reanalysis, reflex testing) from aliquot; mixing studies; platelet aggregation testing; [§] clot waveform analysis (CWA)**						
Time resolution	Sampling can be performed for up to 1800 seconds at intervals of 0.1 seconds						
Measuring time	Up to 1800 seconds for each parameter (automated stopping/extension)						
Incubation	34 wells at 37.0 ±1.0°C						
Number of user-definable (open) channels	80,000						

Sample handling	
Type of sample	Primary tubes and/or secondary sample cups
Sampling mechanism	Automated sample and standard predilution. Up to 64 samples processed simultaneously.
Pre-analytical variables check (PSI™ technology)	Assay-based qualitative check for HIL and proper fill of primary tube. System detects interferences and makes automated actions based on predefined settings.
Traceability of results	Operator name, test date/time, reagent, washing solution/consumables lot information, reagent elapsed time onboard, HIL and sample-volume checks, test reaction position, testing and reagent table temperatures, test protocol number, test measurement time, dilution ratio, QC performed date/time, calibration curve identification, and maintenance/error logs
Carrier system	Four continuous-access sample racks with various specific tube holders and adapters available
Maximum load	24 samples when front-loading; 4 racks with 6 samples per rack; 120 samples with optional sampler; 20 racks with 6 samples per rack

Sample handling (continued)			
STAT sample loading	One priority position with cap piercing and bar-code identification. STAT samples supersede routine samples.		
Storage temperature	Room temperature		
Storage racks	Onboard, post-testing storage capacity of up to five sample racks		
Handling	Flexible mix of various capped and/or uncapped sample tubes and 4 mL conical sample cup		
Primary sample probe	Liquid-level sensing, crash protection, clog and bubble detection, liquid surface verification		
Automatic sample predilution	Samples can be diluted (0 to 19:1 to 259) for multidilution analysis (MDA), and mixing tests and can be retained for auto repeat tests		
Cap piercing	Pierces caps on primary tubes; up to 120,000 times		
Secure aliquot technology	One-time cap-piercing per sample tube; multiple tests taken from aliquot		
Reagent handling			
Reagent recognition	Internal reagent identification via 1D and/or 2D bar code enables tracking of individual vials		
Dispensing accuracy	Volume dispensed	Reproducibility	Accuracy
Reagent probe	50 µL	CV 3% or less	48.0 ±3.0 µL
	100 µL	CV 1% or less	99.0 ±2.0 µL
Sample probe	10 µL	CV 5% or less	9.5 ±1.0 µL
	100 µL	CV 1% or less	98.5 ±2.0 µL
Volume	Reagent dispensing pipette: 20–200 µL Sample dispensing pipette: 20–250 µL (combined diluent and sample) Aspirated volumes of sample and diluent are in the range of 4–200 µL		
Dispensing mechanism	4 probes: 2 heated for reagents and 2 for samples, controls, and calibrators (aliquot and sample arm). 4 catchers for sample reaction tube transfers.		
Loading system	Manual placement of reagents; removable reagent trays		
Reagents onboard	38 tilted positions for reagents/controls/calibrators; 6 buffer solution positions		
Storage temperature	38 tilted reagent positions cooled to 4–12°C ±2°C		
Handling	Flexible mix of reagent positions within the reagent table; various adapters available		
Mixing position	Up to seven positions available on the reagent table		
Storage capacity onboard	PT/APTT: 3000 tests; PT/APTT/Fib: >2700 tests		
Onboard stability	Continuous onboard refrigeration, anti-evaporation caps, optional reagent rack with automated reagent caps for selected reagents		
Reagent volume management	Tracks volume and informs operator of reagent levels, number of tests remaining, lot onboard stability, time elapsed after reagent set, vial type, reagent set date and time, calibration and control validity, and expiration date		
Reagent allocation planning	Software automatically predicts number of tests per day and per shift		

§HYPHEN BioMed reagent.
**Research use only.

Throughput†† (tests/hour, approx.)		
	CN-3000	CN-6000
PT	225 (single-parameter analysis)	450
APTT	217 (single-parameter analysis)	450
PT/APTT	215 (simultaneous analysis)	401
PT/APTT/AT/DD	129 (simultaneous analysis)	257
Numbers are for normal-mode processing		
Operation		
Access mode	Continuous random access (samples, reaction tubes)	
Calibration	2–12 point calibration curves with maximum of 5 repeated analyses per point, up to 10 calibration curves per lot; 10 reagent lot groups per assay	
Calibration curve	250 user-defined parameters	
View calibration	Graphical display of calibration curves from up to 10 different reagent lots/parameters	
Auto calibration/ auto QC	User-defined by specified time, time interval, number of sample measurements, or new reagent vial. The number of QC analysis replications can be assigned per assay.	
Temperature control	37°C ±0.5°C for measuring channels, sample incubation area, and reagent probe	
Sample and abnormal reaction monitoring	Coagulation curve abnormalities; overreaction to antigen excess; interfering assay-based HIL; sample volume check; analysis value error monitoring (checks for deviations, replication difference, MDA slope ratio, and results of repeated analyses)	
Walk-away time	CN-3000 Up to 13 hours	CN-6000 Up to 7 hours
Automation connectivity	Point-in-space connectivity to total lab automation or workcell for both systems†	
Reaction tubes		
Type	Single reaction tubes	
Loading	Automatic continuous access; up to 1200 reaction tubes onboard. Waste capacity >1500 reaction tubes. Lot number tracking for reaction tubes.	
Reaction tube stirring	Up to 40 reaction tubes, with stirrer bars onboard	
System fluid		
Cleaning and rinsing	Washing solutions supplied externally; lot number tracking for washing solutions	
System containers	20 L containers for water (rinse), and 10/20 L containers for waste (optional). Optional direct plumbing (water and waste). Optional connectivity to Millipore Elix Essential water purification systems.	
Maintenance		
Hands-on time	Daily: 3–5 minutes Weekly: 5–10 minutes Monthly: <3 minutes	
Onboard maintenance logs	Automatically logs scheduled routine maintenance activities. Notification and reporting via software.	
Computer/printer		
Workstation	PC	
Display	24" wide-screen LCD display with touch operation	
Printer	Graphic printer (optional)	
Input devices	Touchscreen, keyboard, mouse, 2D bar-code reader	
Data storage	About 10,000 samples with a maximum of 60 results per sample	

Software	
LIS interface	ASTM E1381-02, ASTM E1394-97, IEEE802.3
Host connection	Bidirectional RS-232C serial port or via Ethernet-TCP/IP
Operating system	MICROSOFT WINDOWS 10-based
Onboard help	Software-embedded instruction manual; error detection and guidance; application guide available online
Automatic system startup and backup	Yes
System security	Full audit trail, full user-/role-based access management, automatic screen lock, contactless smart card log-in†
Power supply	
Operating voltage	Main unit: 100–240 V; Pneumatic unit: 100–117 V/220–240 V
Power consumption	Main unit: ≤1080 VA; Pneumatic unit (processing): ≤280 VA
Main frequency	50–60 Hz
Environmental conditions	
Operating temperature	15–30°C
Humidity	30–85% (no condensation except on the reagent table)
Atmospheric pressure	70–106 kPa
Waste heat	Approx. 4776 BTU/h (1204 Kcal/h)
Noise level	55 dB or less at standby; 65 dB or less during analysis (excluding sudden noise that stops within 5 seconds and alarms)
Dimensions	
Main unit (includes PC and monitor)	Approx. 720 (W) x 906 (D) x 1350 (H) mm
Pneumatic unit	Approx. 280 (W) x 355 (D) x 400 (H) mm
Optional sampler	Approx. 330 (W) x 1030 (D) x 830 (H) mm
Weight	
Main unit	Approx. 230 kg
Pneumatic unit	Approx. 17 kg
Optional sampler	Approx. 81 kg
Quality control	
X-control, Levey-Jennings control, Westgard multi-rule monitoring. Up to 40 QC files. 50 charts and 1200 data plots per QC file.	
Compliance	
Safety standards	
IEC 61010-2-081:2001+A 1, IEC 61010-2-101:2002, IEC 61010-1:2001, EN 61010-2-081:2002+A 1, EN 61010-2-101:2002, EN 61010-1:2001, IEC 61010-2-081:2015, IEC 61010-2-101:2015, IEC 61010-1:2010, IEC 62471:2006, IEC 62304:2006+A1:2015, EN 61010-1:2010, EN 61010-2-081:2015, EN 61010-2-101:2017, EN 62471:2008, EN 62304:2006+A1:2015, UL 61010-1:2012, UL 61010-2-081:2015, UL 61010-2-101:2015, CAN/CSA-C22.2 NO. 61010-1-12, CAN/CSA-C22.2 NO. 61010-2-081:15, CAN/CSA-C22.2 NO. 61010-2-101:15, CAN/CSA-CE III EC 62304:14+A1:2018	
EMC standards	IEC 61326-2-6:2012, EN 61326-2-6:2013
Ordering information	
Product description	Catalog number
Sysmex CN-3000 instrument kit 230 V	11554330
Sysmex CN-6000 instrument kit 230 V	11554360
Sysmex CN Systems Int accessory kit	11554370
Optional Sampler for CN Systems	11554371

††Throughput values were determined by the time to first result; processing capability varies depending on the reagent used. Throughput values stated above were determined using Sysmex study protocol with PT (Thromborel S Reagent), APTT (Pathromtin SL Reagent), INNOVANCE D-Dimer Reagent, and AT (INNOVANCE AT Reagent) test applications.

†Under development. Not available for sale. Future availability cannot be guaranteed.

At Siemens Healthineers, our purpose is to enable healthcare providers to increase value by empowering them on their journey toward expanding precision medicine, transforming care delivery, and improving patient experience, all made possible by digitalizing healthcare.

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
An estimated 5 million patients globally benefit every day from our innovative technologies and services in the areas of diagnostic and therapeutic imaging, laboratory diagnostics, and molecular medicine, as well as digital health and enterprise services.

We are a leading medical technology company with over 120 years of experience and 18,000 patents globally. Through the dedication of more than 50,000 colleagues in 75 countries, we will continue to innovate and shape the future of healthcare.

**This brochure is intended for use outside the U.S. only (OUS).
Display or promotion to U.S. audiences is prohibited.**

Actin, Berichrom, Dade, INNOVANCE, Innovin, Pathromtin, ProC, PSI, Thromboclotin, Thromborel, and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. Sysmex is a trademark of the Sysmex Corporation. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Siemens Healthineers Headquarters
Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
[REDACTED]
siemens-healthineers.com

Legal Manufacturer
Sysmex Corporation
1-5-1 Wakinohama-Kaigandori,
Chuo-ku,
Kobe 651-0073, Japan

Legal Manufacturer
HYPHEN BioMed
155 rue d'Eragny
95000 Neuville-sur-Oise
France

for the Products
PSI technology

for the Products
ADP, Epinephrine, Arachidonic
Acid, Ristocetin, Collagen

Distributed by
Siemens Healthcare Diagnostics
Products GmbH
Laboratory Diagnostics
Emil-von-Behring-Strasse 76
35041 Marburg, Germany

Legal Manufacturer
for all other products
Siemens Healthcare Diagnostics
Products GmbH
Laboratory Diagnostics
Emil-von-Behring-Strasse 76
35041 Marburg, Germany

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