

VIVIX-S 1717N Specifications



© Vieworks. 2017 All rights reserved.

Under copyright laws, this manual may not be reproduced, in whole or in part, without the written permission of Vieworks.

The specifications and related information in this manual may be changed without notice. Refer to Vieworks Download System (VDS) for the latest version of manuals.

Contents

1. Instruction	3
1.1 Document Guide	3
1.1.1 Target.....	3
1.1.2 Symbols.....	3
1.1.3 Notations.....	3
1.2 Revision History	4
1.3 Contact Department	4
2. Products	5
2.1 Detector	5
2.1.1 Specifications.....	5
2.1.2 Drawing Sheet.....	6
2.1.3 Functions	8
2.1.4 Wireless Communication.....	9
2.1.5 Use Environment.....	10
2.2 SCU	10
2.2.1 FXRS-02A	10
2.2.2 FXRS-03A	11
2.2.3 FXRS-04A	11
2.2.4 FXRP-02A.....	12
2.3 Battery Pack (FXRB-03A)	13
2.4 Battery Charger (FXRC-02A)	14
2.5 Battery Charger (FXRC-03A)	15
3. Performance	17
3.1 FXRD-1717NA(W) (Csl)	17
3.2 FXRD-1717NB(W) (GOS)	18
4. Defect	19
4.1 Defect Type	19
4.2 Defect Allowance.....	19
5. Regulatory Information	20
5.1 Medical Equipment Classification	20
5.2 Product Safety Standard.....	20

1. Instruction

1.1 Document Guide

1.1.1 Target

This document is intended for customers who use the **VIVIX-S 1717N** detector.

1.1.2 Symbols

Before attempting to use Vieworks' product, refer to the information in this specifications. It is important for you to read and understand the contents of specifications for operating the products safely.

Caution and Warning



- This symbol is used to indicate a potentially hazardous situation which may cause death, personal injury or substantial property damage if the instructions are ignored. Be sure to understand the instructions of this symbol for the safe operation.

Information



- This symbol is used to indicate reference and complementary information related to the product. You should read the instructions of this symbol carefully.

1.1.3 Notations

Bold Types

We applied bold font style to the words which indicated products terms, or the words and sentences which are needed to transmit clear meaning to the customers. This helps you to easily distinguish the words from other technical ones for explaining functions.

1.2 Revision History

Ver.	Date	Descriptions
1.1	2015-09-22	Initial Release
		(Modified) 2 Products
1.2	2015-10-02	(Modified) 3 Performance (Modified) 5 Regulatory Information
1.3	2016-04-28	(Added) 2 Products
1.4	2016-07-01	(Added) 2 Products
1.5	2016-07-27	(Changed) 2 Products (Changed) 5 Regulatory Information
1.6	2016-11-04	(Changed) 2 Products
1.7	2017-03-09	(Added) 2.1.4 Wireless Communication (Added) 2.2 SCU

1.3 Contact Department

For any comments or inquiries regarding this document, contact via email below.

Item	Contents
Department	Customer Support Team in Vieworks
E-mail	CustomerSupport@vieworks.com

2. Products


2.1 Detector

2.1.1 Specifications

Item	Specifications
Model	<ul style="list-style-type: none"> FXRD-1717NA (CsI) FXRD-1717NB (Gadox) FXRD-1717NAW (CsI) FXRD-1717NBW (Gadox)
Image Sensor	<ul style="list-style-type: none"> TFT: a-Si (Amorphous Silicon)
X-ray Scintillator type	<ul style="list-style-type: none"> FXRD-1717NA(W): CsI: TI (Thallium doped Caesium Iodide) FXRD-1717NB(W): Gd₂O₂S:Tb (Gadolinium oxysulfide)
Pixel Pitch	<ul style="list-style-type: none"> 0.14mm (140μm)
Field of View	<ul style="list-style-type: none"> 17" x 17"
Active Area (H x V)	<ul style="list-style-type: none"> 430.08mm x 430.08mm
Active Array	<ul style="list-style-type: none"> 3072 x 3072 pixels
Effective Area	<ul style="list-style-type: none"> 428.4mm x 428.4mm
Effective Array	<ul style="list-style-type: none"> 3060 x 3060 Pixels
Grayscale	<ul style="list-style-type: none"> 16 bit
Spatial Resolution	<ul style="list-style-type: none"> Min. 3.5 lp/mm
Image Acquisition Time (Wired)	<ul style="list-style-type: none"> 1.5 sec.
Image Acquisition Time (Wireless)	<ul style="list-style-type: none"> 3 sec.
Capacity for Image Backup	<ul style="list-style-type: none"> Max. 200 images
Recommended Cycle Time	<ul style="list-style-type: none"> 15 sec.
X-ray Synchronization Control	<ul style="list-style-type: none"> AED (Auto Exposure Detection) DR Trigger (External line trigger)
Rated Power Supply	<ul style="list-style-type: none"> DC +24V, Max. 1A ▫ Wired: Powered by SCU with a tether interface cable. ▫ Wireless: Powered by a battery pack (3,100 mA h X 2)
Power Consumption	<ul style="list-style-type: none"> Max. 24W
Dimensions (H x W x D)	<ul style="list-style-type: none"> 460mm x 460mm x 15.5mm
Weight	<ul style="list-style-type: none"> 4.2kg (Wired) 4.3kg (Wireless / without a battery pack)
Image Transfer	<ul style="list-style-type: none"> Wired: Gigabit Ethernet (1000BASE-T) via PoE (Power over Ethernet) Wireless: IEEE802.11n
Data Transmission Rate (Wired)	<ul style="list-style-type: none"> Max. 1Gbps
Data Transmission Rate (Wireless)	<ul style="list-style-type: none"> Max. 300Mbps (MIMO 2X2)

2.1.2 Drawing Sheet

FXRD-1717NA / 1717NB (Wired Detector with connecting a tether interface cable)

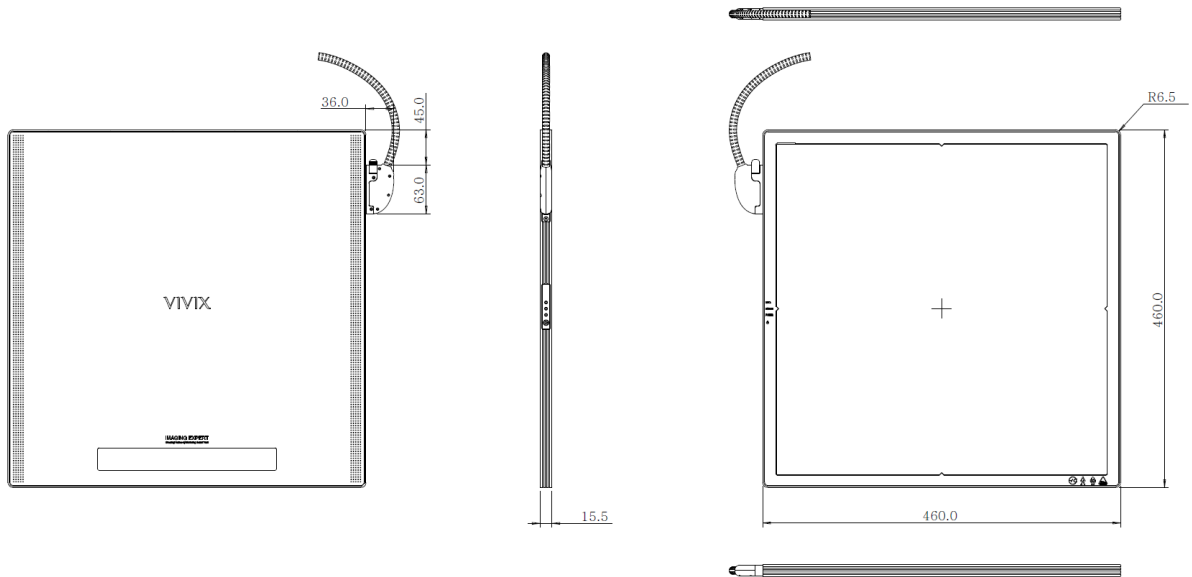


Item	Description
Dimensions (H × W × D)	460.0mm × 460.0mm × 15.5mm
Curvature of Edges	R6.5



- The allowed tolerance of thickness of a detector is from **-2.0mm** ~ **+0.5mm**. (Under the **ISO4090** regulation).

FXRD-1717NAW / 1717NBW (Wireless Detector)

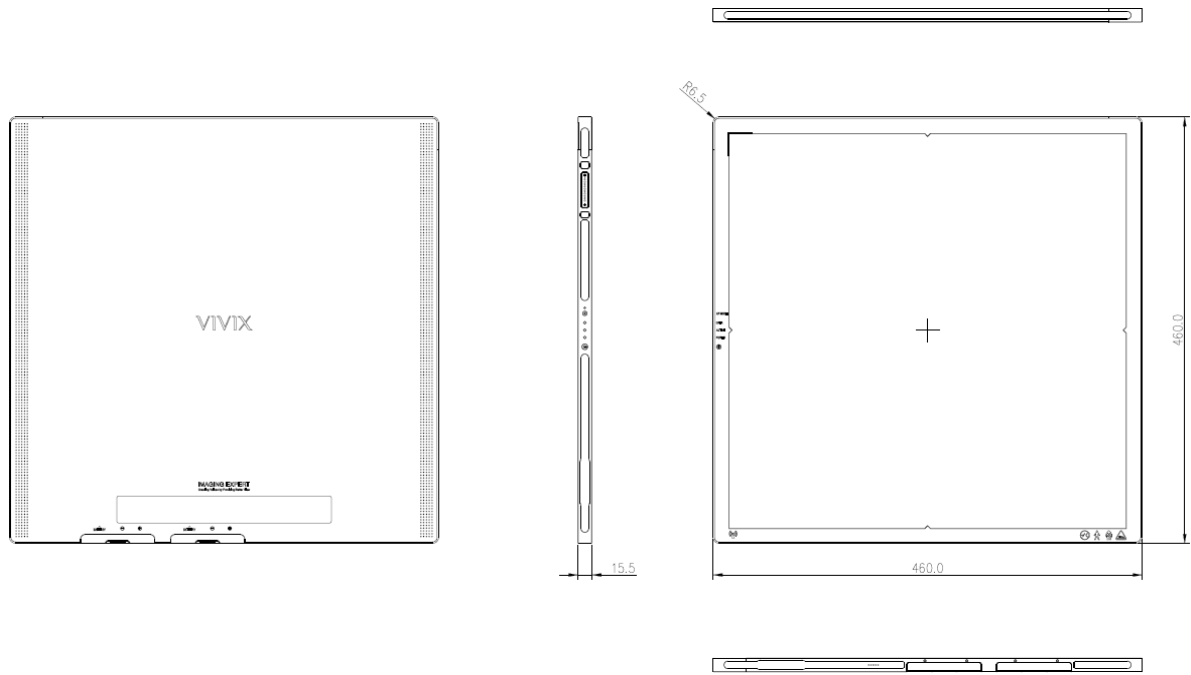


Item	Description
Dimensions (H × W × D)	460.0mm × 460.0mm × 15.5mm
Curvature of Edges	R6.5



- The allowed tolerance of thickness of a detector is from **-2.0mm ~ +0.5mm**. (Under the **ISO4090** regulation).


2.1.3 Functions

FXRD-1717NA / 1717NB (Wired)

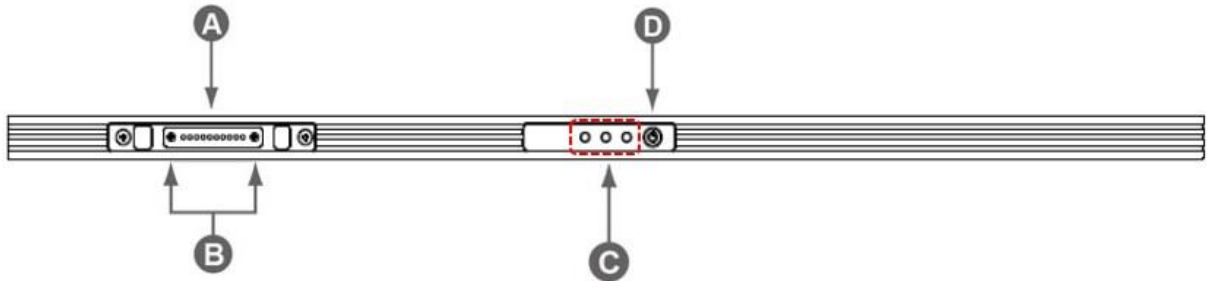

Name	Description
A Tether interface connector	Used for tighten the tether interface cable. • Used for wired connection between a detector and SCU.
B Tether interface holder	Fix/release holder of the tether interface cable.
C Status indicator	Detector status indicator • DATA LED (Blue): Indicates communication and transmission • ACTIVE LED (Orange): Indicates ready to work • POWER LED (Green): Indicates power On/Off status
D Power button	Detector power button

FXRD-1717NAW / 1717NBW (Wireless)

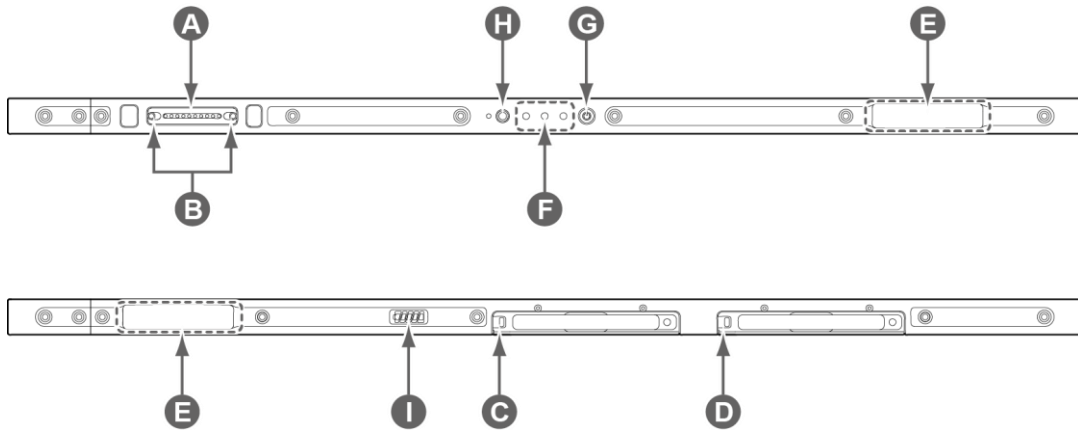

Name	Description
A Tether interface connector	Used for tighten the tether interface cable. <ul style="list-style-type: none"> Used for wired connection between a detector and SCU.
B Tether interface holder	Fix/release holder of the tether interface cable.
C Battery 1 lock/unlock lever	Applied for a FXRB-03A battery pack.
D Battery 2 lock/unlock lever	Applied for a FXRB-03A battery pack.
E Antenna for wireless LAN	Antennas for wireless communication (2EA) Detector status indicator
F Status indicator	<ul style="list-style-type: none"> DATA LED (Blue): Indicates communication and transmission ACTIVE LED (Orange): Indicates ready to work POWER LED (Green): Indicates power On/Off status
G Power button	Detector power button
H AP ON /OFF Button	Turns on / off the AP mode.
I Battery Remaining LED	Notices the remaining of battery in 5 levels.

2.1.4 Wireless Communication

Item	Specifications
Wireless standard	IEEE802.11n
Frequency range	2.412 ~ 2.472GHz (13 Channels) 5.18 ~ 5.24GHz (4 Channels), 5.745 ~ 5.805GHz (4 Channels)
Data transmission rate	802.11n: Max. 300Mbps (MIMO 2X2)
Modulation	BPSK, QPSK, 16-QAM, 64-QAM
Transmission power	Max. 17dBm
Security	WPA2-PSK
Antenna	Dual Band Antennas (2EA, inbuilt)



• The specification of detector and SCU are same, but the location of antenna is different.

2.1.5 Use Environment

Item	Operation	Storage & Transportation
Temperature	+10 ~ +35°C	-15 ~ +55°C
Humidity	30 ~ 85% (Non-condensing)	10 ~ 90% (Non-condensing)
Atmospheric pressure	700 ~ 1060hPa	500 ~ 1060hPa
Shock	1.6G	20G
Vibration	0.7G	0.7G
Drop limit	500mm	500mm


2.2 SCU

2.2.1 FXRS-02A

Specifications

Item	Specifications
Model	<ul style="list-style-type: none"> FXRS-02A
Power Supply	<ul style="list-style-type: none"> Input: AC100 to 240V, 50/60Hz, Max. 2.0-0.8A Output: DC +24V 3.25A, 78W
Cable Connection port	<ul style="list-style-type: none"> Gigabit Ethernet ports (3ea) Two(2) PoE (Power over Ethernet) ports
Dimension (H × W × D)	<ul style="list-style-type: none"> 210.0 mm × 300.0 mm × 54.2 mm
Weight	<ul style="list-style-type: none"> 2.5 kg

Drawing Sheet

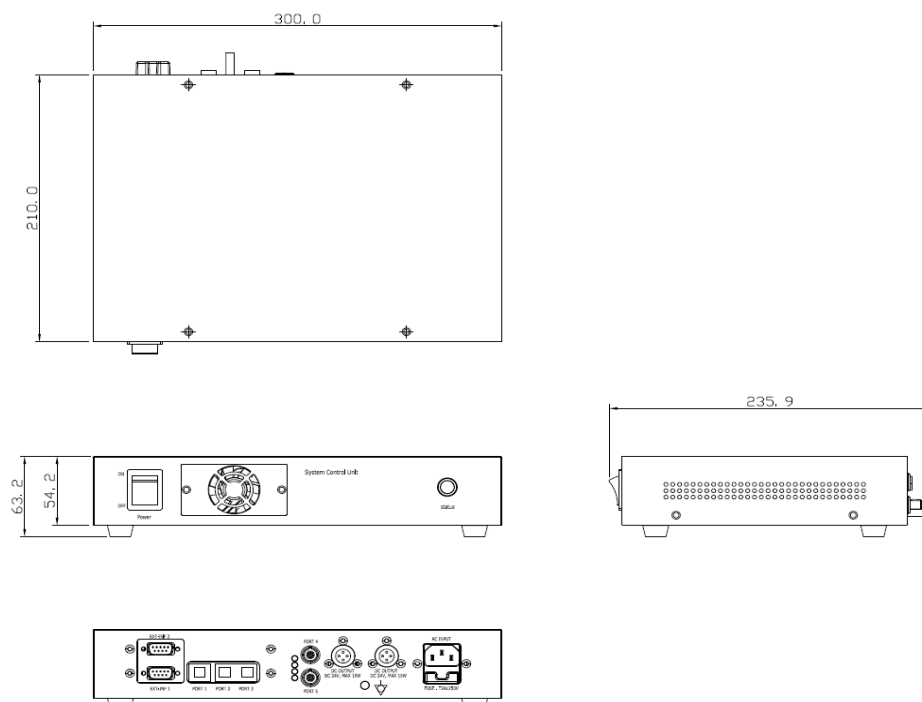


2.2.2 FXRS-03A

Specifications

Item	Specifications
Model	• FXRS-03A
Power supply	• Input: AC100 to 240V, 50/60Hz, Max. 2.0-0.8A • Output: DC +24V 3.25A, 78W
Cable connection port	• Gigabit Ethernet port (3EA) • PoE (Power over Ethernet) Port (2EA)
Wireless communication	• IEEE 802.11n (2.4 GHz / 5 GHz)
Dimension (H × W × D)	• 236.0mm × 300.0mm × 58.0mm
Antenna	• 140mm (2EA, Dual band)
Weight	• 2.8 kg

Drawing Sheet


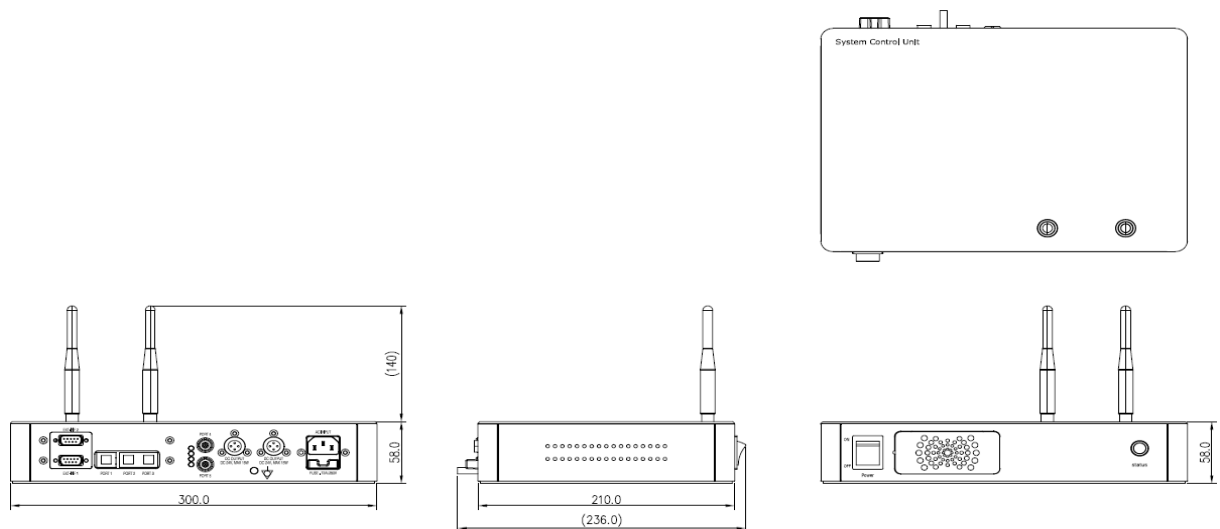


2.2.3 FXRS-04A

Specifications

Item	Specifications
Model	• FXRS-04A
Power supply	• Input: DC +24V 2A Max
Cable connection port	• Gigabit Ethernet port (3EA) • PoE (Power over Ethernet) Port (1EA)
Wireless communication	• IEEE 802.11n (2.4 GHz / 5 GHz)
Dimension (H × W × D)	• 210.0mm × 170.0mm × 45.0mm
Antenna	• 140mm (2EA, Dual band)
Weight	• 1.2kg

Drawing Sheet

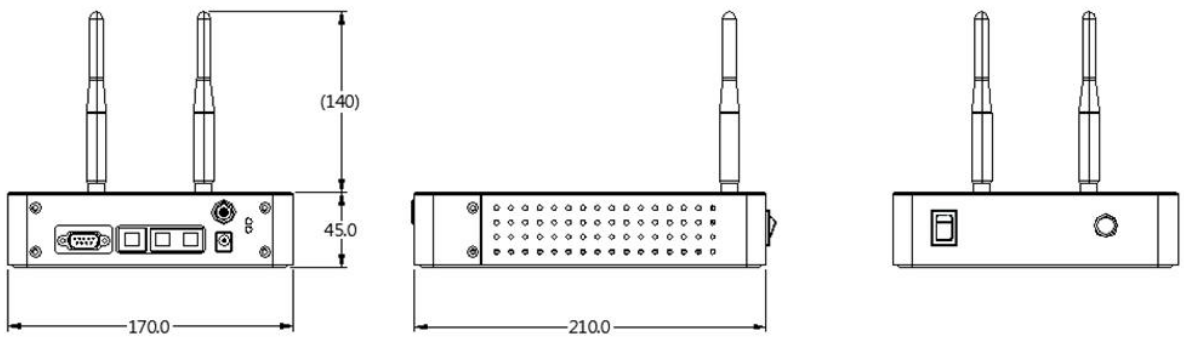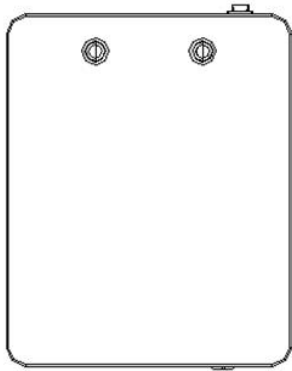


2.2.4 FXRP-02A

Specifications

Item	Specifications
Model	<ul style="list-style-type: none"> FXRP-02A
Power supply	<ul style="list-style-type: none"> Input: DC +24V 1.0A Max.
Cable connection port	<ul style="list-style-type: none"> Gigabit Ethernet port (1EA) PoE (Power over Ethernet) Port (1EA)
Dimension (H × W × D)	<ul style="list-style-type: none"> 108 mm × 109 mm × 29.5 mm
Weight	<ul style="list-style-type: none"> 330g

Drawing Sheet

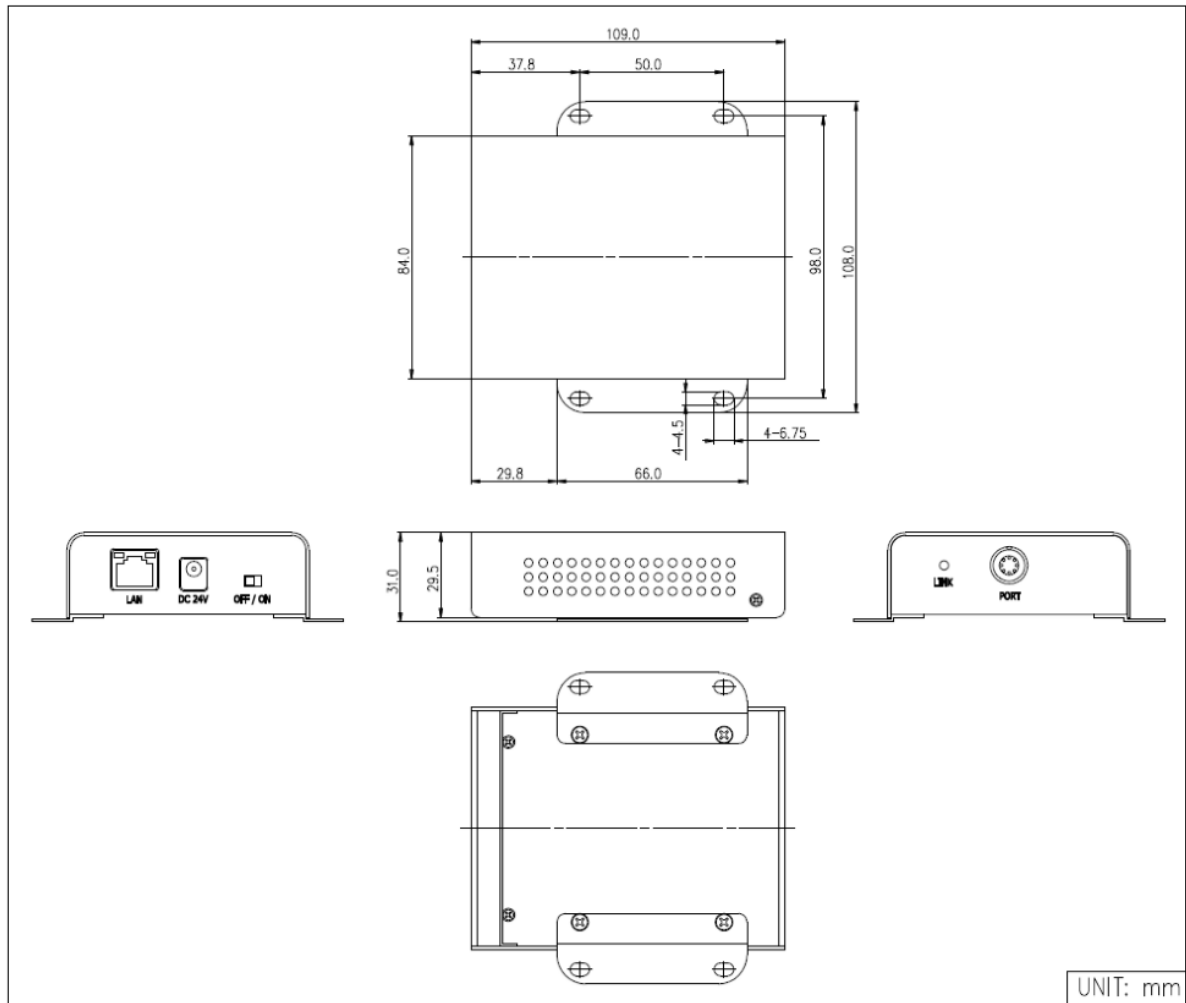

2.3 Battery Pack (FXRB-03A)


Specifications

Item	Specifications
Model	FXRB-03A
Type	Lithium Ion
Rated Power Supply	Output: DC +7.6V
Capacity	3,100mAh
Number of Cell	2S1P (2 Series 1 Parallel)
Operation Time	6 hours (when sleep mode is off)
Life	Approx. 500 times (Fully charged/Discharged completely, 1 cycle)
Dimension (H × W × D)	160.0mm × 61.8mm × 5.7mm
Weight	115g



- The battery operation time increases under the sleep mode depending on the operational condition and environment.

Drawing Sheet

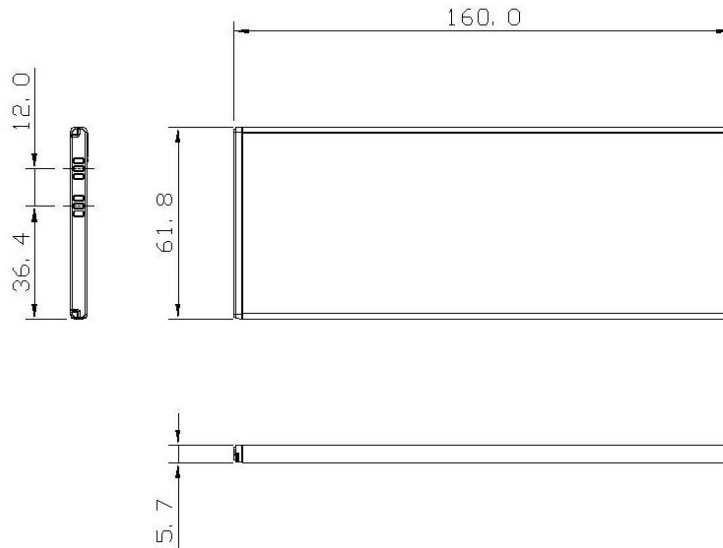


2.4 Battery Charger (FXRC-02A)

Specifications

Item	Specifications
Model	FXRC-02A
Simultaneous Charging	2 battery packs
Charging time	2 1/2 hours
Rated power supply	DC +24V, 2A Max.
Dimension (H × W × D)	192.0mm × 101.0mm × 26.0mm
Weight	0.8 kg

Drawing Sheet

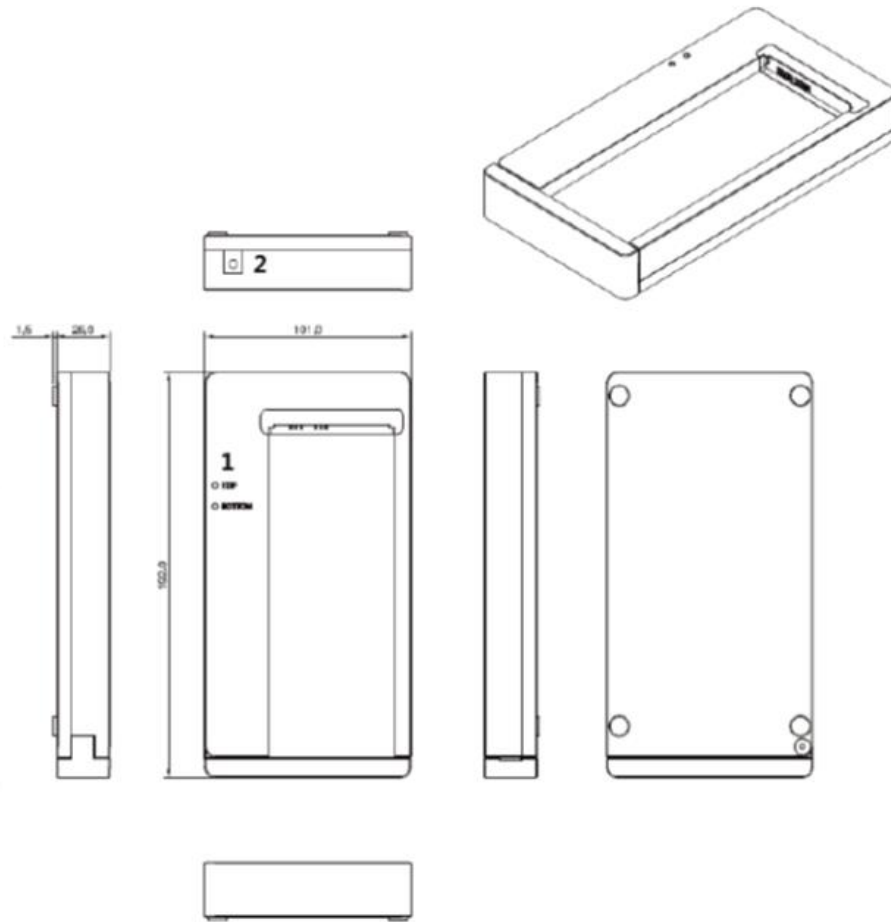


No.	Name	Description
1	Top / Bottom LED	Indicates the position and status of a battery being charged.
2	Power input port	Supplies electric power by connecting a power adapter to the battery charger.

2.5 Battery Charger (FXRC-03A)

Specifications

Item	Specifications
Model	FXRC-03A
Simultaneous Charging	2 battery packs (FXRB-01A (1ea) / FXRB-03A (1ea))
Charging time	FXRB-01A: 3 hours FXRB-03A: 2 1/2 hours
Rated power supply	DC +24V, 2A Max.
Dimension (H × W × D)	163.6 mm × 190 mm × 34.0 mm
Weight	0.5 kg

Drawing Sheet

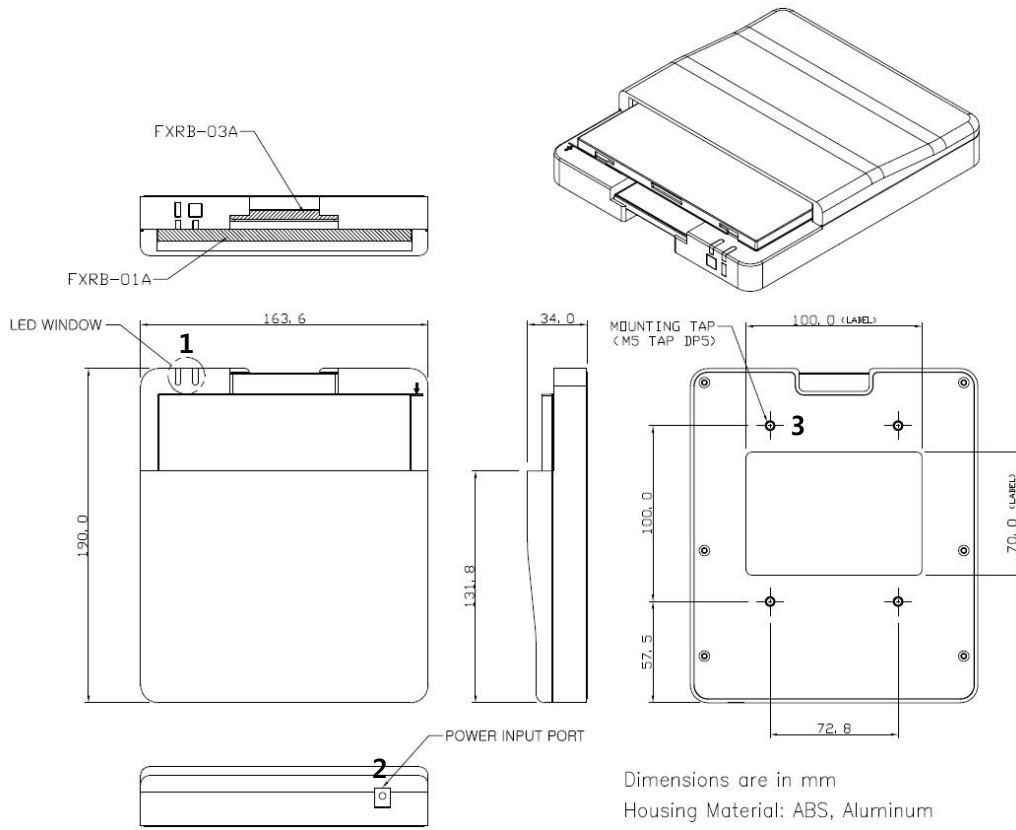

No.	Name	Description
1	LED Window	Indicates the location and status of a battery pack being charged. <div style="display: flex; justify-content: space-around; margin-top: 5px;"> <input type="checkbox"/> : FXRB-03A <input type="checkbox"/> : FXRB-01A </div>
2	Power input port	Supplies electric power by connecting a power adapter to the charger.
3	Mounting hall	Screw holes for fixing the charger to the external device.

3. Performance

3.1 FXRD-1717NA(W) (CsI)

- Test Condition: RQA5, 2.5uGy, IEC 62220-1 Standard
- The typical values are for reference only.

Parameters	Unit	Minimum	Typical	Maximum
Dark Noise	cts	-	4	5
Offset (Black Image)	cts	800	-	1600
Sensitivity at G=1	cts/uGy	550	600	650
Quantum Limited Dose	uGy	-	-	0.2
Signal to Noise Ratio	dB	17	-	-
Max. Exposure Level	uGy	90	-	-
Dynamic Range	a.u	450	-	-
MTF	0.5 lp/mm	87	90	-
	1 lp/mm	68	72	-
	2 lp/mm	36	40	-
	3 lp/mm	19	22	-
DQE	0.5 lp/mm	52	55	-
	1 lp/mm	47	50	-
	2 lp/mm	34	40	-
	3 lp/mm	22	26	-



- The formula of dynamic range is as follows;
 - $Dynamic\ Range = \frac{Max.Exposure\ Level}{Quantum\ Limited\ Dose}$

3.2 FXRD-1717NB(W) (GOS)

- Test Condition: RQA5, 2.5uGy, IEC 62220-1 Standard
- The typical values are for reference only.

Parameters	Unit	Minimum	Typical	Maximum
Dark Noise	cts	-	4	5
Offset (Black Image)	cts	800	-	1600
Sensitivity at G=1	cts/uGy	450	500	550
Quantum Limited Dose	uGy	-	-	0.3
Signal to Noise Ratio	dB	17	-	-
Max. Exposure Level	uGy	110	-	-
Dynamic Range	a.u	400	-	-
MTF	0.5 lp/mm	81	83	-
	1 lp/mm	58	60	-
	2 lp/mm	24	26	-
	3 lp/mm	9	11	-
DQE	0.5 lp/mm	30	33	-
	1 lp/mm	25	27	-
	2 lp/mm	16	18	-
	3 lp/mm	6	9	-




- The formula of dynamic range is as follows;

$$\square \text{ Dynamic Range} = \frac{\text{Max.Exposure Level}}{\text{Quantum Limited Dose}}$$

4. Defect

4.1 Defect Type

Type	Description
Single Defect	Isolated defects, adjacent pixels are normal. (Class 0)
Cluster Defect	More than consecutive 2 pixels are defected. (Class 1~Class 8)
Line Defect	Defect occur horizontal direction from left to right, or vertical direction from top to bottom.

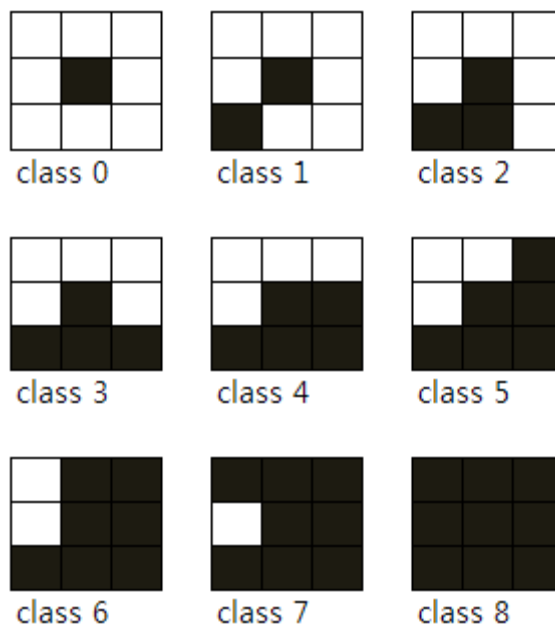

- No cluster defects are allowed over 3x3 pixels.

4.2 Defect Allowance

Item	Unit	Value
Total number of pixel defects	cts	Max. 20,000 pixels
Number of line defects	cts	Max. 5 lines
Number of normal lines between two bad lines	cts	Min. 3 lines

5. Regulatory Information

5.1 Medical Equipment Classification

Item	
Type of protection against electrical shock	Wireless Model: Class I or Internally Powered Wired Model: Class I
Degree of protection against electrical shock	Type B
Degree of protection against ingress of water	IP53 (Degrees of protection against ingress of water and dust provided by enclosure.)
Operation mode	Continuous operation
Flammable anesthetics	NOT suitable for use in the presence of a flammable anesthetic mixture with air or oxygen or nitrous oxide.

5.2 Product Safety Standard

USA / Canada

Item	
IEC 60601-1(ed.2 am1+ am2+ co1)	Medical electrical equipment- Part1: General requirements for safety
UL 60601-1(ed.2)	-
CSA-C22.2 No. 601-1-M90 (R2006)	Medical electrical equipment – Part 1: General requirements for safety (adopted amendment 2:1995 to IEC60601-1)
IEC 60601-1-2: 2007(ed.3)	Medical electrical equipment-Part 1-2: Collateral standard: Electromagnetic compatibility
IEC 60601-1-4: 2000(ed.1.1)	Medical electrical equipment- Part 1-4: Collateral Standard: Programmable electrical medical systems
IEC 62304:2006	Medical device software-software life cycle processes
ISO 14971:2012	Medical Device- Application of risk management to medical devices

European Union

Item	
MDD (Medical Device Directive)	93/42/EEC as amended by 2007/47/EC
EN ISO 13485:2012	Medical devices – Quality Management systems – Requirements for regulatory purposes
EN 60601-1: 2007(ed.3)	Medical electrical equipment- Part1: General requirements for safety
IEC 60601-1-2: 2007(ed.3)	Medical electrical equipment-Part 1-2: Collateral Standard : Electromagnetic compatibility-Requirements and tests
IEC 60601-1-4: 2000(ed.1.1)	Medical electrical equipment- Part 1-4: Collateral Standard : Programmable electrical medical systems
IEC 62304:2006	Medical device software-Software life cycle processes
ISO 14971: 2012	Medical device – Application of risk management to medical devices.

VIEWWORKS

Viewworks.Co.,Ltd

(Gwangyang-dong) 41-3, Burim-ro 170beon-gil, Dongan-gu, Anyang-si,
Gyeonggi-do, 431-060 Republic of Korea



Telephone: +82-70-7011-6161

Fax: +82-31-386-8631

Homepage: <http://www.viewworks.com>

European representative: DONGBANG ACUPRIME

1 Forrest Units, Hennock Road East, Marsh Barton, Exeter EX2 8RU, UK



Telephone: +44(0)-1392-829500

Homepage: <http://www.acuprime.com>
